

# PANAVIA™ SA Cement Universal

---

**SIMPLE, CLEAN AND VERSATILE**

FORGET ABOUT PRIMERS



# SIMPLE, CLEAN AND VERSATILE

The MDP and LCSi monomer new chemistry:

- Provides high bond strength
- Eliminates additional steps



Forget about the primer process and additional steps for your daily needs.




The Original MDP monomer creates a strong chemical bond to Metal Alloy, Zirconia, Enamel and Dentin.



The Original LCSi monomer creates a strong chemical bond to Porcelain, Lithium Disilicate and Composite Resin.

## Easy Cementation Without Additional Primers

Say goodbye to complicated bonding protocols and simplify your work flow.

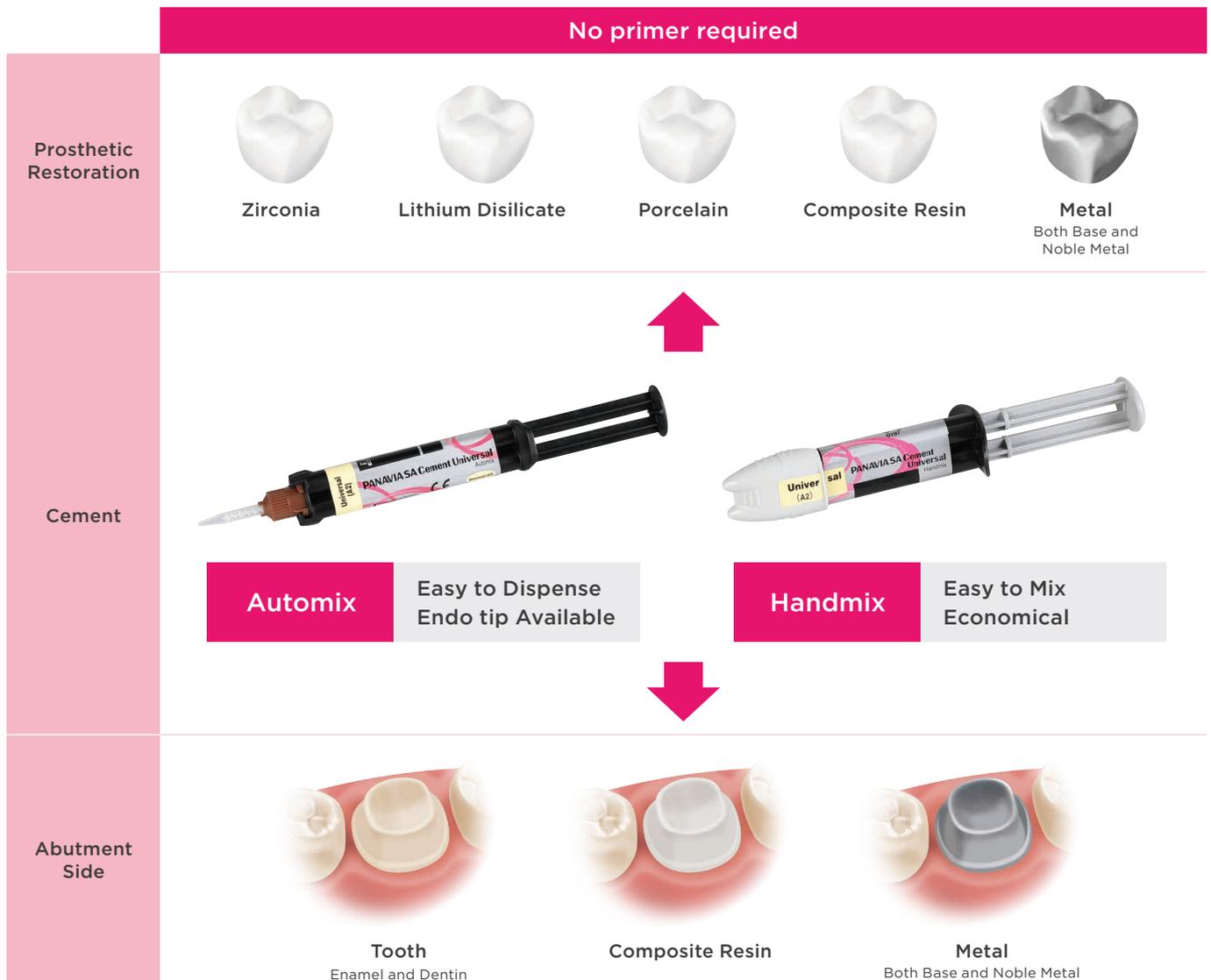

# Clean: Fast & Easy Clean-Up

Two options of polymerization, the same easy clean-up result.



2 to 5 seconds

2 to 4 minutes




When the cement is used in combination with CLEARFIL™ Universal Bond Quick, the setting time of the excess cement will be faster (1 to 2 seconds).

Easy clean-up that is gentle to the gums and minimizes the risk of accidental ingestion.

<b>F-ion release</b>	<b>Storage at Room Temperature</b>	<b>Radiopacity</b>	<b>Film thickness</b>
Contains surface treated sodium fluoride	2-25 °C / 36-77 °F	Yes	Approx. 16 µm

## Shades

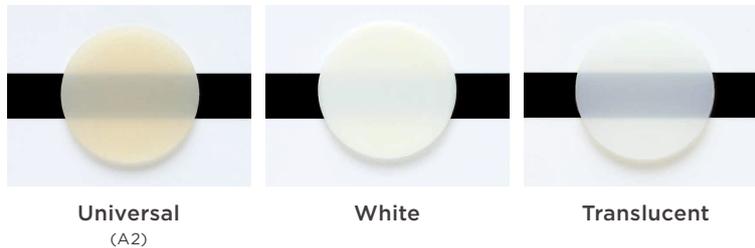


Universal (A2)

White


Translucent

# Original MDP Monomer: High Bond Strength

## Shear Bond Strength to Restorative Materials

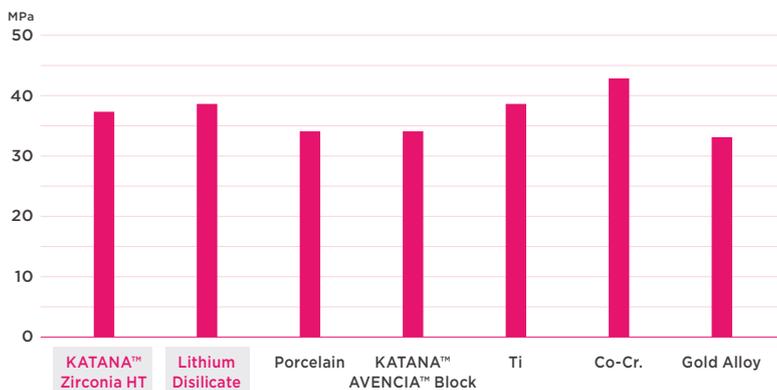

## Shear Bond Strength to Tooth Structure

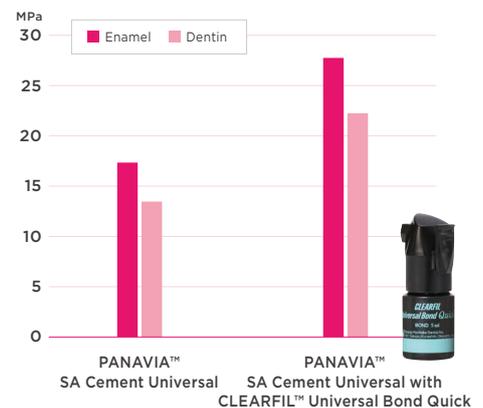

CLEARFIL™ Universal Bond Quick can be used on the tooth structure to enhance the bond strength for cases with poor retention.

# One single procedure for your crown cementations

Zirconia, Lithium Disilicate or Metal. No primers required.  
One single procedure regardless of the restorative material.



**Conditioning the crown.\***  
\* Refer to "Conditioning the prosthetic restoration".



Dispense an equal amount of Paste A & B.  
Mix Paste A & B for 10 seconds.



Apply the cement to the crown.



Place the crown.




Light-cure for 2 to 5 seconds or chemical-cure for 2 to 4 minutes, then remove the excess cement.



Maintain isolation for 5 minutes.\*  
\* For a translucent restoration, light-cure.

## Conditioning the prosthetic restoration

Follow the instructions for use of the restoration material.  
In the absence of specific instructions, we recommend the following procedure:

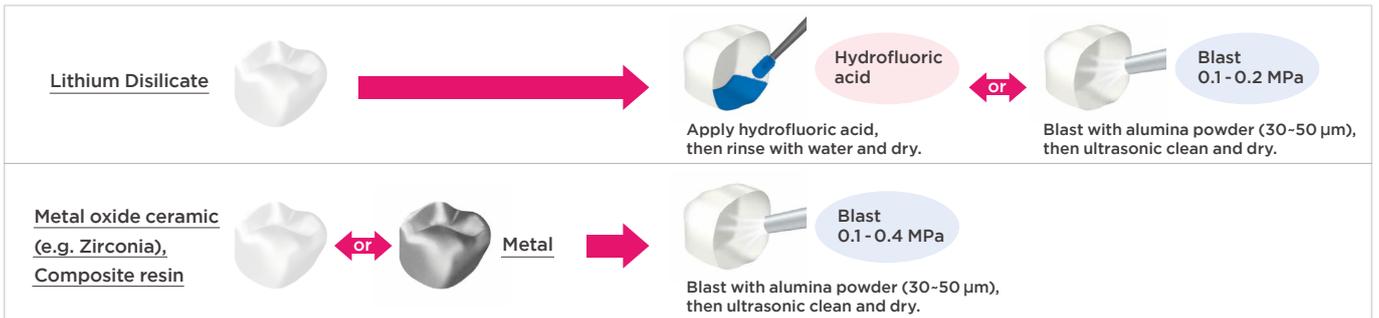

### PANAVIA™ SA Cement Universal Automix

- Universal (A2)
  - White
  - Translucent
- Content: 1 syringe:  
Paste A and B: 8.2 g (4.6 mL), Mixing Tip: 20 tips



### PANAVIA™ SA Cement Universal Handmix

- Universal (A2)
  - White
  - Translucent
- Content: 1 syringe:  
Paste A and B: 9.2 g (5 mL)



**Kuraray Noritake Dental Inc.**

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

Website [www.kuraraynoritake.com](http://www.kuraraynoritake.com)

- Before using this product, be sure to read the Instructions for Use supplied with the product.
- The specifications and appearance of the product are subject to change without notice.
- Printed color can be slightly different from actual color.

"PANAVIA", "AVENCIA" and "CLEARFIL" are trademarks of KURARAY CO., LTD  
"KATANA" is a trademark of NORITAKE CO., LIMITED.